QuickVue[®] iFOB TEST

QUIDEL

SPECIMEN COLLECTION AND RETURN KIT

Patient Instructions


For assistance with these instructions contact:

- Your physician's office
- or -
- Quidel Technical Support - 800.874.1517
Monday through Friday, from 7:00 a.m. to 5:00 p.m.,
Pacific Time, e-mail: technicalsupport@quidel.com


COLLECTION KIT CONTENTS

Before using this kit, please make sure it contains all the necessary items

Return Mail Box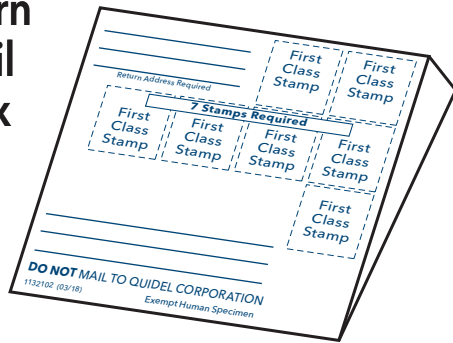


Collection Paper

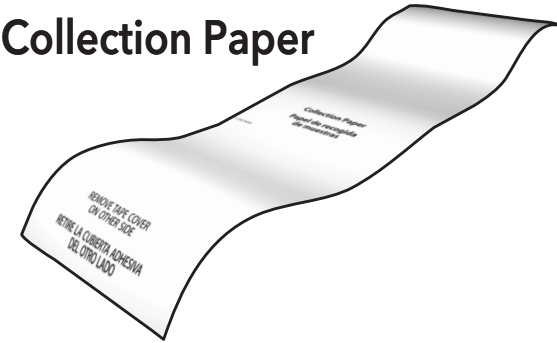
Specimen Pouch

Absorbent Sleeve
(save for mailing)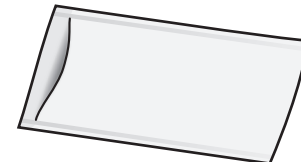


Specimen Collection Tube

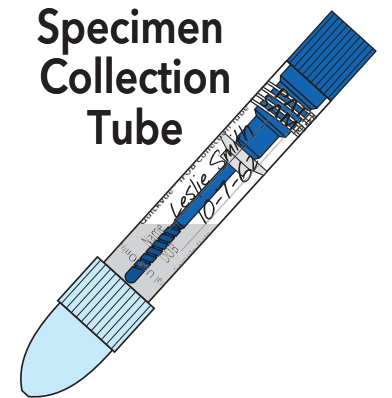1 BEFORE YOU BEGIN:

- Fill in Sample Date on Specimen Collection Tube label. If Name and DOB are blank, also fill in that information.
- Do not collect specimen if bleeding is present from hemorrhoids, constipation, urination or menstruation.
- Urinate prior to positioning Collection Paper.
- Do not allow specimen to come in contact with toilet water.
- Do not urinate on fecal specimen or Collection Paper.

A. Remove tape cover from each end of the Collection Paper.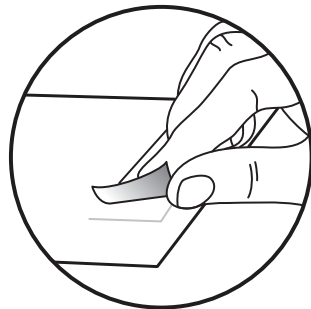

B. Tape paper on rear half of toilet bowl as shown. Allow paper to sag.


C. Deposit stool on paper. **DO NOT** allow stool to contact toilet water.


D. With the fecal specimen still on the Collection Paper, proceed to Step 2.


2 A. Unscrew Sampler.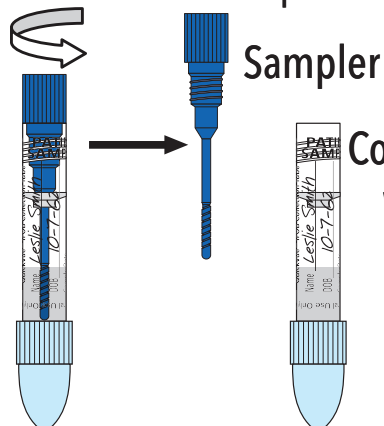

B. Pierce specimen in at least **5** different sites.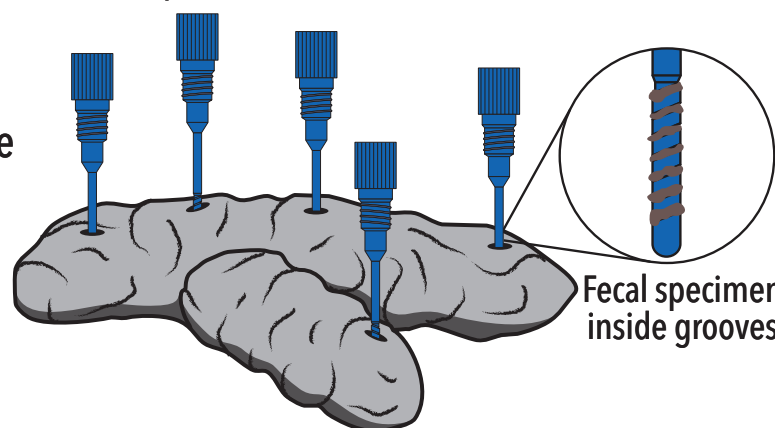

C. Insert Sampler into Collection Tube, firmly tighten. Shake the Tube well.

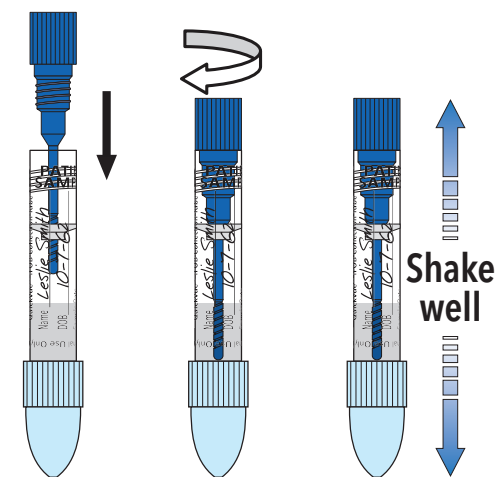
D. Flush remaining specimen and Collection Paper.


3 A. Insert Collection Tube into Absorbent Sleeve.


B. Insert Absorbent Sleeve into Specimen Pouch, and seal Pouch.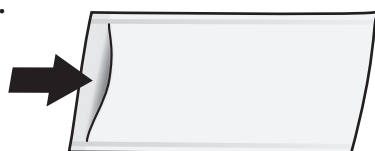

C. Insert Specimen Pouch and lab slip/test order, if applicable, into Return Mail Box and seal Box.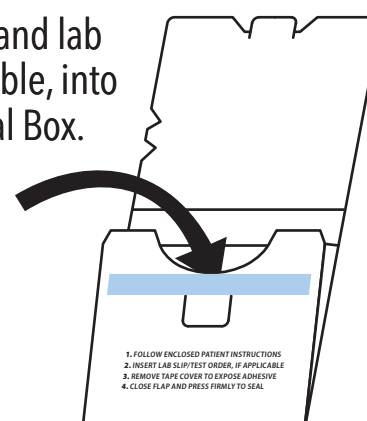

D. Complete the "to" and "from" addresses, if blank. Apply seven 1st Class stamps and mail the sealed Return Mail Box to your physician's office or lab. **DO NOT MAIL TO QUIDEL CORPORATION**

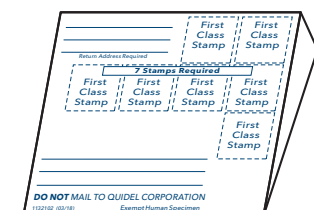MAIL

Remove tape cover to seal Box

Best if mailed within 24 hours of specimen collection 1133207 (03/18)

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